



Instrument Engineering Change Notice

NO. DRS2000-2201

Sakura Finetek Japan Co. Ltd.

PRODUCT NAME : Tissue-Tek DRS 2000 DATE : June 14, 2022
 PRODUCT CODES : All models ORIGINATOR : Otsuka
 IECN TYPE : PART DESIGN CHANGE REFERENCE NUMBER : 20042-1, 20550
 SERIAL NUMBERS AFFECTED : See Remarks ATTACHMENTS : AI0-IF-013(ENG)-00
 FIELD RETROFIT REQUIRED : NO

Title: Change of Gas Damper, Release of Service Manual Amendment

[REASON AND DESCRIPTION FOR CHANGE]

Due to production discontinuation, the gas damper for opening and closing the cover has been replaced with the new gas spring which is comparable in dimensions and gas pressure.

(NOTE) The old and new parts are interchangeable. However, special attention needs to be paid to the direction of the new gas spring at installation to avoid interference with the neighboring parts. Refer to the DRS2000 Service Manual AMENDMENT (AI0-IF-013(ENG)-00) for replacement.

[PART CHANGE] (IVDP=Inventory Disposal, ICHB=Inter Changeable)

Description	Old	QTY	IVDP	New	QTY	ICHB
Gas Damper / Y0075	D9280014	2	NO	---	--	YES
Gas Spring / TGS281-100-55 (15x6)	---	--		00001-5055	2	YES
DRS2000 Service Manual AMENDMENT				A65-IF-013(ENG)-00		

[REMARKS]

J-B1: 4928 0210 and after JC-D2:49460072 and after
 J-B2: 4930 0190 and after E-B2: 4930 0084 and after
 J-D1: 4929 0090 and after E-D2: 4932 0811 and after
 J-D2: 4931 0421 and after

[DISTRIBUTION]

SFA
 SFJ
 SFE
 SCN

[APPROVAL]

Name : T. Sugawara

Signature :