



Instrument Engineering Change Notice

NO. VIP6AI-2001

Sakura Finetek Japan Co. Ltd.

| | | | |
|-------------------------|--|------------------|---|
| PRODUCT NAME | : Tissue-Tek VIP6 AI | DATE | : March 18, 2020 |
| PRODUCT CODES | : 6040, 6041, 6042, 6043 | ORIGINATOR | : A. Ito, T. Iijima |
| IECN TYPE | : SOFTWARE CHANGE AND PART DESIGN CHANGE | REFERENCE NUMBER | : 16713, 16757-1 |
| SERIAL NUMBERS AFFECTED | : See Remarks | ATTACHMENTS | : O79-900-09VIP6GUT_AI_O79_900.zip, O79-900-09UPDATE.zip |
| FIELD RETROFIT REQUIRED | : NO | | |

Title: Software and Hardware Changes for New CI

[REASON AND DESCRIPTION FOR CHANGE]


(1) The logo displayed at the top left corner of each operation screen was deleted due to the change to the new CI.

The loading screen was also changed with the change of the initial program loader, as follows:

* The initial program loader is not supplied for field service.



New loading screen (white background)

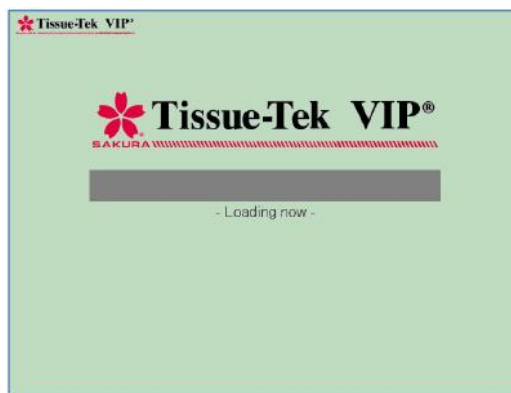

Old loading screen



New operation screen



Old operation screen

(2) The logo on the control panel cover was changed to the new CI logo. The cover with the old logo is supplied as a service part.

[PART CHANGE] (IVDP=Inventory Disposal, ICHB=Inter Changeable)

| Description | Old | QTY | IVDP | New | QTY | ICHB |
|--------------------------|------------|-----|------|------------------|-----|------|
| Control Program | O79-900-08 | 1 | | O79-900-09 | 1 | YES |
| Control Panel Cover (CI) | --- | --- | | A-MO79-0300-00-A | 1 | NO |
| Control Panel Cover | O79-310-00 | 1 | NO | --- | --- | NO |

(Note) If the new control program O79-900-09 is installed to the system with an older version of the program, the loading screen is NOT updated although the operation screen is updated. This does not affect functions and performance of the system.

[REMARKS]

A1: 6040 0631 and after
J0: 6041 0445 and after
E2: 6042 0669 and after
J2: 6043 0180 and after

[DISTRIBUTION]

SFA

SFJ

SFE

[APPROVAL]

Name : T. Sugawara

Signature :

